


SELF-SWAB INSTRUCTIONS

FOR ABBOT PANBIO RAPID ANTIGEN TEST



DO NOT PROCEED IF:

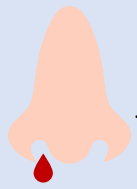

Nosebleed in the past **24 hours**



Nasal surgery in the last **4 weeks**



Facial injury in the last **8 weeks**

SWABBING PROCEDURE

1

Please clean your hands before starting the test.




2

Before starting the test, please take out one swab stick, one test device, one buffer bottle and one extraction tube. Take the test device out from its packaging.



3

Take the cap off the buffer bottle. Hold the buffer bottle vertically and fill the extraction tube with the buffer fluid until it flows up to the Fill-line of the extraction tube.

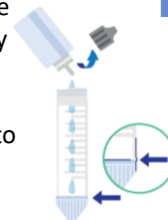

4

Remove swab stick from package.



5

Insert the swab stick into the nostril (~2cm) while gently rotating. Twirl the stick 5 times.



6

Repeat for the other nostril with the same swab.




7

Swirl the swab tip in the buffer fluid against the walls of the extraction tube for at least 5 times.




8

Squeeze the reagent tube when extracting the swab.

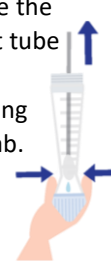

9

Break the swab at the breakpoint and leave the swab in the tube. Close the cap of extraction tube.

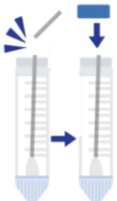

10

Use your finger to gently tap the side of the tube a few times. This helps to mix the solution.




11

Open the nozzle cap at the bottom of the extraction tube.



12


Squeeze 5 drops of the solution into the specimen well on the test device. Close the nozzle cap once you are done.

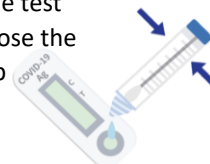

13

Wait for 15mins before checking the results on the test cartridge. Please keep the cartridge on a flat surface during this time.

15 mins

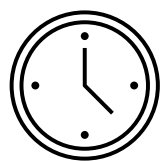

14

Read the results on the test cartridge. **If you see a line appearing at both the "C" and the "T" position, this indicates a positive test. Please seek medical attention for further evaluation and confirmatory testing.**

If you see a line appearing at only the "C" position, this indicates a negative test.

If you see a line appearing at only the "T" position, the test is invalid. Please repeat the test on another test kit.

15

Dispose all used swab sticks, reagent tube and test cartridge into the biohazard bag.



16


Clean your hands after the test.




POSITIVE



NEGATIVE

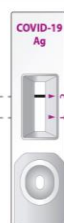

INVALID

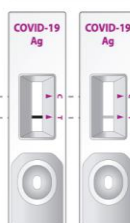

"C" Control Line
"T" Test Line